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**RIGID ORTHOSIS FOR REST,  
DUIMBRACE NACHT,  
ORTESI DA RIPOSO,  
RUHE-DAUMENBANDAGE**

Ref19542-B - 11/2023

**CE** MILLET INNOVATION

MILLET Innovation ZA Champgrand  
26270 Loriol/Drôme - FRANCE

contact@epitact.com  
www.epitact.com

FRANÇAIS

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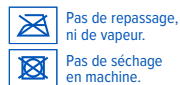
ENGLISH

**INDICATIONS:** During the night or in any situation in which you are not using your hand, regularly wearing the EPITACT® rigid thumb orthosis for rest helps relieve joint pains at the base of the thumb and helps restrict development of rhizarthrosis

**CONTRA-INDICATIONS:** Not for use on cutaneous lesions.

**PRECAUTIONS FOR USE:** Wash before first use. In case of severe oedema: monitor the hand during use. Remove the device if excessively tight and hampering circulation. If in doubt, ask your doctor's or pharmacist's advice.

Do not apply any cream or ointment before putting on the orthosis.  
Do not use near a flame or incandescent material.

**FEATURES:** The EPITACT® orthosis immobilises the thumb in resting position, opening the commissure between the index finger and thumb. The alignment of the bones limits the forces applied to the trapeziometacarpal joint and relieves pain.

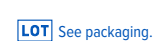
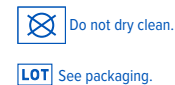
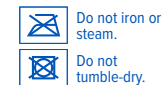
**ERGONOMICS:** The immobilisation orthosis offers you optimum comfort owing to its reinforcements modelled to adapt perfectly to the morphology of your hand. Light and extra thin, you will forget you are wearing it.

**TIP:** To supplement the rigid resting orthosis, we recommend that you wear the EPITACT® flexible activity orthosis in all situations in which you need to use your hand.

**LIFESPAN :** Wash after use. Hand washing at 30°C, with a mild soap, without detergent or bleach. Rinse well. Squeeze to expel moisture. Do not expose while drying to a source of direct heat (radiator, sunlight). Do not use any stain removers. Discard and replace the device as soon as fabric or seam damages occur.

**COMPOSITION :** 57% textile (Polyamide, Elasthan, Polyester), 43% TPE thermoplastic elastomer.

In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority. Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2017.

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