# GENOUILLÈRE ROTULIENNE

PATELLAR KNEE BRACE,  
KNEEBANDAGE KNIESCHIJJ,  
GINOCCHIERA PER ROTULEA,  
PATELLA-KNEEBANDAGE

231101 - ref. 19587-B

MILLET Innovation ZA Champgrand  
26270 Loriol/Drôme - FRANCE

contact@epitact.com  
www.epitact.com

## FRANÇAIS

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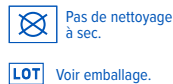
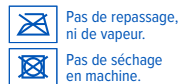
## ENGLISH

**INDICATIONS:** The EPITACT® patellar knee brace is recommended for painful and inflammatory syndromes in the knee and in particular, patellar syndromes. It is indicated for the prevention of kneecap instability.

**CONTRA-INDICATIONS:** Not for use in the event of arteritis, significant varicose condition, consequences of phlebitis or oedema (lymphatic/venous) in the leg and/or thigh so as not to obstruct circulation. Not for use on cutaneous lesions.

**PRECAUTIONS FOR USE:** It is recommended that you wash your device before first use. To ensure a perfect hold, it is essential to wash the silicone surface of the thigh elastic with soap and water after each use. Do not use while immobile (only use while active). If in doubt, ask your doctor or your pharmacist, or any other competent professional for advice. This medical device is a regulated healthcare product which bears CE marking, according to legislation. Assessment of compliance has been performed by the ASQUAL.

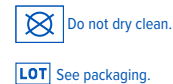
**FEATURES:** The EPITACT® knee support has a patented patellar EPITHELIUMFLEX® tendon which is positioned around the kneecap to limit lateral movements. The 3D elastic weave exerts pressure which enhances sensory perception in the joint (proprioception) and helps to stabilize it. The flexible lateral ribs reinforce the knee support.

**ERGONOMICS:** With a silicone elastic strip on the thigh, the knee support does not slide and has excellent hold. With a patented cushion pad behind the knee (peroneal fold), it is not uncomfortable and does not irritate. The kneecap is protected from pressure with a decompression weave zone. The micro-aerated stitching enables the release of sweat for optimum comfort.

**TIP:** If you are an athlete, when resuming activity, we recommend you use another product from the EPITACT® range: the PHYSIOstrap™ Sport to secure your fragile knee.

**LIFESPAN:** Wash after use at 30°C, with a mild soap, without detergent or bleach. Rinse well. Squeeze to expel moisture. Do not expose while drying to a source of direct heat (radiator, sunlight). Do not use any stain removers. Discard and replace the device as soon as fabric or seam damages occur.

**COMPOSITION :** 67,8% textil (Polyamide, Elasthan, Polyester), 22,7% Aluminium, 9,5% Silicone. In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority. Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2017.

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