ALMOFADAS PLANTARES com Epithelium 26®

INDICAÇÕES

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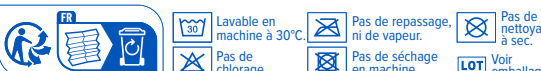
PLANTAR CUSHIONS with Epithelium 26®

INDICATIONS

Prevent and alleviate forefoot pain.

CONTRAINDICATIONS

- Do not use on a skin lesion.
- Persons suffering from arthritis, diabetes, lower limb neuropathy or varicose ulcers: never use the product on a wound, monitor the condition of your foot during use, and in all cases seek advice from your doctor or chiropodist. The strap may cause friction between the two toes. This solution should therefore be avoided by diabetics suffering from neuropathy.

PRECAUTIONS FOR USE

- Wash before first use.
- Wear next to the skin.
- The printed part of the cloth should be in contact with the sole of the foot. Do not turn over.
- Slip your foot into the cushion and place the retaining tab between your big toe and your second toe.
- To help the cushion remain in position, wear tights, socks or stockings over it.

WHAT IT IS USED FOR

- Patented by chiropodists, Epithelium 26® is an effective substitute for the sole of the foot. This silicone gel's mechanical properties allow it to replace the missing plantar tissue that is causing pain: it is remarkably effective in redistributing your body weight over the whole of your foot.
- The presence of Epithelium 26® in the plantar area allows Plantar Cushions to alleviate pain effectively.

CLEANING

- Washing after use at 30°C in the washing bag supplied or wash by hand (without rubbing).
- Discard and replace the device as soon as fabric or seam damages occur.
- Leave to dry naturally. Do not iron.

COMPOSITION: 80% silicone, 20% polyamide/elastane fabric.

In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority.

Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2001.

Machine washable at 30°C.	Do not iron or steam.	Do not dry clean.
Do not bleach.	Do not tumble-dry.	See packaging.

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