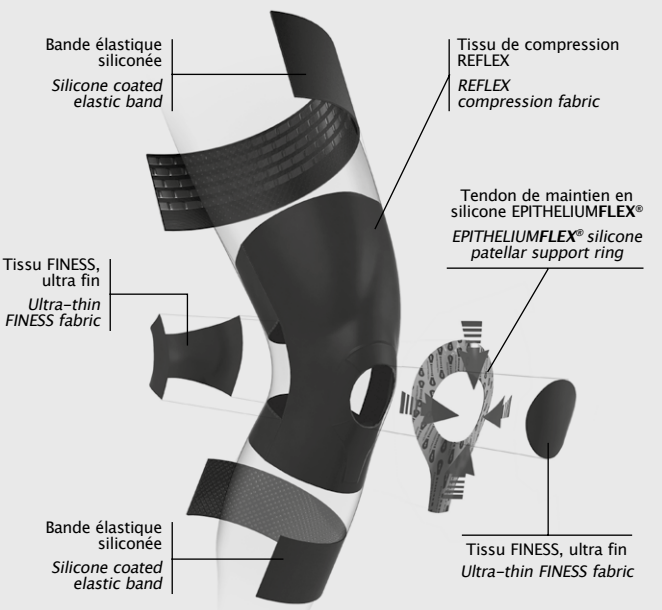
PATENTED TECHNOLOGY

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ENGLISH PHYSIOOstrap™ EPIHELUMFLEX®01

INDICATIONS: Wearing the EPIHELUMFLEX®01 PHYSIOOstrap™ helps relieve patellar conditions (patellar syndrome, patellar instability, patellar tendonitis, etc.). In the event of a knee injury with residual effects (ligament weakness, osteoarthritis, etc.), it helps stabilise the joint when resuming sports activities.

CONTRAINDICATIONS: To ensure blood circulation is not impeded, do not use in the event of arteritis, severe varicose conditions or residual effects of phlebitis or oedema (lymphatic/venous) of the lower leg and/or thigh. Do not use with compression hosiery. Do not use on an open wound.

PRECAUTIONS FOR USE:
- To avoid any risks of irritation generated by the silicone elastic bands, the PHYSIOOstrap™ should not be worn in static positions, particularly with the wearer lying down (sleep) or sitting (including bus or car journeys, etc.). The PHYSIOOstrap™ should be put on immediately before activity, to be worn during movement, during sports activities.
- If you use calf compression sleeves (such as Booster), place the lower elastic band of the knee brace over the sleeve.
- Place the PHYSIOOstrap™ on the thigh without stretching.

HOLD:
- Do not apply moisturising products before using the PHYSIOOstrap™ to ensure optimum hold.
- If you encounter any hold problems in extreme conditions (sports with sudden changes of speed, running on steep slopes or over long distances, high temperatures), we recommend that you wear a thigh sleeve over the PHYSIOOstrap™.
- To guarantee good hold, the product must be washed before each use. At the very least, wash the silicone elastic bands with soap and water (without rubbing) to remove any dirt.

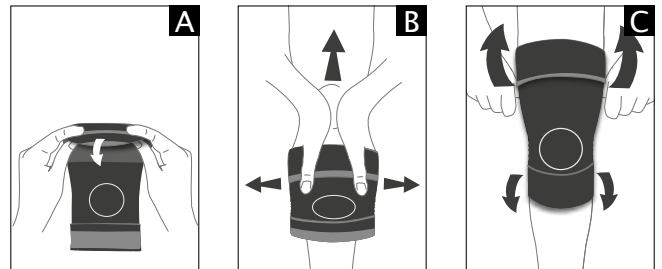
SPECIFIC: The PHYSIOOstrap™ works dynamically, without impeding movement. It can therefore be used preventively since it maintains muscle strength.

ERGONOMIC: Concentrated technology weighing less than 60g, the EPIHELUMFLEX®01 PHYSIOOstrap™ has been specially designed for sports. The EPIHELUMFLEX®01 PHYSIOOstrap™ is designed to fit either the right or the left knee. It does not slip and does not cause any discomfort at the back of the knee (popliteal fossa). Extra-thin, it can be worn under all types of clothing.

WASHING INSTRUCTIONS: Wash after use. Machine wash at 30°C or hand wash (without rubbing), leave to dry naturally. Do not iron. Discard and replace the device as soon as cracks in the gel or fabric or seam damages occur.

COMPOSITION : 70% polyamide/elastane fabric, 30% silicone.
In case of pain or any other unusual reaction, contact your doctor. The silicone tendon in contact with the skin may cause mechanical irritations, such as blisters (< 0.1%). Any major incidents involving the device must be reported to the manufacturer and the relevant health authority.
If you experience pain, oedema, discomfort or any other abnormal reaction, stop using the product and consult a doctor.
MILLET Innovation cannot be held liable in cases where the manufacturer's recommendations have not been followed during use. French is the reference language. CE initial marking: 2014.

- Machine washable at 30°C.
- Do not iron or steam.
- Do not dry clean.
- Do not bleach.
- Do not tumble-dry.
- See packaging.

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EN HOW TO FIT
A) Unfold the elastic bands outwards.
B) Fold the PHYSIOOstrap™ back on itself to slip it on and position the kneecap in the centre of the "support ring".
C) Unfold the elastic bands without stretching the PHYSIOOstrap™ on your leg.
D) Adjust your PHYSIOOstrap™ in the standing position and ensure that the support rings are in a flat position.

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