SEPARADORES com EPITHELIUM™

INDICAÇÕES

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SEPARATORS with EPITHELIUM™

INDICATIONS

EPITHELIUM™ separators are used to separate two toes between which there is excessive pressure and friction.

CONTRAINDICATIONS

- Do not use on a skin lesion.
- Persons with arteritis, diabetes, polyneuritis, peripheral neuropathy, or foot ulcers: consult your physician or chiropodist before application and monitor your foot during use.

PRECAUTIONS FOR USE

- Wash before first use.
 - Place 1 separator between two toes. The widest part should be located under the foot, with the flat part against the corn.
- Note: To widely separate two toes:
- Position two separators against one another.
 - Place the assembly obtained between the toes.
- Once the metatarsal corn has been removed by the podiatrist, and healing, a separator is put in place. This system prevents the keratome from automatically reforming. The wider part can be placed either at the top or at the bottom. Choose according to the comfort you feel.

WHAT IT IS USED FOR

- The separator spreads the toes apart by a few millimetres. It is made of EPITHELIUM™ which has remarkable properties that distribute pressure and prevent friction.
- The flexible material and the shape of the separator, which match the foot, provide immediate comfort.

CLEANING

- Wash after use. Protections may be washed with soap and water.
- Discard and replace the device as soon as cracks in the gel appear.
- Do not rub, allow to air-dry.

COMPOSITION: 100% silicone.

In case of pain or any other unusual reaction, contact your doctor. Silicone in contact with the skin may cause mechanical irritations, such as blisters (< 0.1%). Any major incidents involving the device must be reported to the manufacturer and the relevant health authority.

MILLET Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2004.

	Hand washable.		Do not iron or steam.		Do not dry clean.
	Do not bleach.		Do not tumble-dry.		See packaging.

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