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CARP'IMMO®

RIGID RESTING WRIST ORTHOSIS | IMMOBILISERENDE
POLSBRACE, ORTESI POLSO RIGIDA DA REPOUSO |
STARRE HANDGELENK-RUHEBANDAGE

Ref19544-B - 11/2023

CE MILLET INNOVATION

MILLET Innovation ZA Champgrand
26270 Lorioi/Drôme - FRANCE

contact@epitact.com
www.epitact.com

FRANÇAIS

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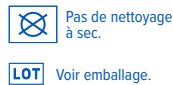
ENGLISH

INDICATIONS: Regularly wearing the CARP'IMMO™ RIGID RESTING WRIST BRACE at night (or at rest) is recommended for those suffering from carpal tunnel syndrome, musculoskeletal disorders, swelling or wrist tendonitis.

CONTRAINDICATIONS: Use only on clean, intact skin.

PRECAUTIONS FOR USE: Wash before first use.

FEATURES: The ultra-light EPITACT™ brace adapts perfectly to your wrist and comfortably holds it in a neutral position, thanks to the integrated semi-rigid lining.

ERGONOMICS: The CARP'IMMO™ brace adapts perfectly to the shape of your hand due to the design of the integrated semi-rigid lining. Lightweight and ultra-thin, you won't even know you're wearing it! The brace opens easily to accommodate your hand and the strap allows you to adjust it perfectly to your wrist.

WASHING INSTRUCTIONS: Wash after use. Hand-wash at 30°C with mild soap (avoid detergent and bleach). Rinse well and squeeze out excess water. It should be dried in an open-air area, not in direct sunlight and away from any heat source. Do not use stain removers or solvents. Discard and replace the device as soon as fabric or seam damages occur.

COMPOSITION : 59% fabric (polyamide, Elastane, polyurethane) 41% thermoplastic elastomer

PROVIDES MAXIMUM SUPPORT AND COMFORT

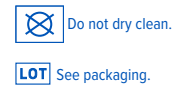
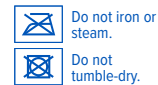
EASILY ADJUSTABLE STRAP
> Adjustable compression around the wrist.

SOFT, ULTRA-THIN FABRIC
> To ensure optimal comfort.

REINFORCED LINING
> Wrist held in a neutral, resting position.

In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority. Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2018.

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