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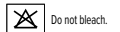
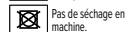
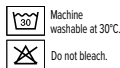
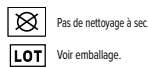
INDICATIONS: Mild to moderate sprains, chronic ankle instability, resuming sporting activity.

CONTRA-INDICATIONS: Do not use if you suffer from arteritis, diabetes, polyneuritis, neuropathy of the lower limbs.

USAGE PRECAUTIONS: Wash before first use. Do not place ERGOstrap™ SPORT in direct contact with the skin. ERGOstrap™ SPORT must be worn over socks, high enough to avoid causing discomfort during wear. Do not use when standing (only use while carrying out your activity). If you use compression sleeves on your calves, place them below ERGOstrap™ SPORT. Do not use with compression stockings.

CARE: Machine-wash at 30°C or hand-wash without rubbing. Wash after use. Discard and replace the device as soon as cracks in the gel or fabric or seam damages occur. Leave to air-dry. Do not iron. Do not expose to heat (> 50°C).

COMPOSITION: 50% polyamide/elastane fabric, 32% heat-mouldable plastic, 18% silicone.

DETAILS: ERGOstrap™ SPORT supports and stabilises the ankle without restricting movement, thanks to the combined action of heat-mouldable supports around the malleoli and EPITHELIUMFLEX® patented strapping.

ERGONOMICS: A technological marvel weighing less than 120 g, ERGOstrap™ SPORT has been specially designed for sport. Perfectly adapted to your body shape and extra-slim (no added thickness under the heel), it is very comfortable and can be worn under any type of sock. You will completely forget about it during your sporting activity.

DIRECTIONS FOR USE:

• Heat-moulding: please follow the attached heat-moulding instructions or watch the heat-moulding video which can be found on www.ergostrapsport.com. ERGOstrap™ Sport can be heat-moulded a several times if necessary.

• Positioning: watch the positioning video which can be found on www.ergostrapsport.com.

In case of pain or any other unusual reaction, contact your doctor. Any major incidents involving the device must be reported to the manufacturer and the relevant health authority. MILLET Innovation cannot be held liable in cases where the manufacturer's recommendations have not been followed during use. French is the reference language. CE initial marking: 2017.

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